

Ankle Fixed Foot Drop Brace

踝骨固定足下垂支具

Model: TD-TV

Features:

- Plastic buckle, durable, convenient and quick to use
- The high-density sponge is soft and comfortable, effectively preventing direct contact between the soles of the feet and plastic
- Velcro strap design, can be adjusted according to personal needs
- Suitable for: foot restoration, foot drop, etc.
- 塑膠搭扣, 耐用性強, 不易損壞, 使用方便快捷
- 高密度海綿柔軟舒適, 有效防止腳底和塑膠件的直接接觸, 避免接觸而產生的不適
- 魔術貼綁帶設計, 可根據個人需要進行調節
- 可拆卸式設計, 舒適透氣面料緊密貼合足部
- 適用於: 足部恢復、足下垂等

Specifications

Product Model 產品型號	TD-TV
Product Function 產品功能	rehabilitation training / fitness 康復訓練 健身
Material 材質	resin plastic board + sponge pad 樹脂塑膠板+海綿墊
Bottom length 底長	28cm
Tube Height 筒高	43.5cm
Model 型號	universal 左右通用
Suitable Size 適合尺碼	38-42碼 38-42size
Product Size 產品尺寸	28*34*11cm
Product Weight 產品重量	520g
Package Size 包裝尺寸	40*30*20cm
Package Weight 包裝重量	1kg



Instructions

- 1) 將產品內裡海綿墊取下
- 2) 將支撐板螺釘取下, 在腳跟處對準孔位將螺釘安裝進去, 腳跟外部將支撐板對準螺孔安裝螺釘旋鈕, 旋轉至不動為止
- 3) 然後將腳墊安裝需求墊在護具前端, 安裝上海綿墊即可穿戴

◆ 注意事項:

1. 如使用後皮膚出現過敏請立即停止使用並尋求醫生診治
2. 必須佩戴舒適, 並不影響血液循環為本
3. 如使用後情況沒有改善, 請尋求醫生協助
4. 清洗時可用清潔劑清洗即可
5. 腳底支撐板安裝時請一定將