

Thumb wrist fixed protective sleeve

拇指腕部固定保護套

Model: TD-IH

Features:

- Artificial curve design, strengthened fixation, double protection
- Versatile left and right, high-adhesive Velcro, easy to fix, adjustable elasticity, no longer worry about finger thickness and wrist size, easy to wear
- Reinforced double fixation straps to protect the injured hand twice and promote hand rehabilitation
- Lightweight fabric, comfortable and breathable
- Suitable for: mother hand, tenosynovitis, thumb injury, keyboard hand
- 人工學曲線設計,加強固定,雙重保護
- 左右通用,高粘魔術貼,輕鬆固定,鬆緊可調,不再為手指粗 細和手腕大小而煩惱,穿戴方便
- 加固雙重固定帶,讓受傷的手二次保護,促進手部康復
- 輕薄面料,舒適透氣
- 適用於:媽媽手、腱鞘炎、拇指受傷、鍵盤手



Specifications

Product Model 產品型號	TD-IH
Product Material 產品材質	composite cloth, aluminum alloy metal strip 複合布 鉛合金金屬條
Model 型號	Free size 均碼
Color 顏色	Royal Blue 寶藍色
Part of use 使用部位	fingers 手指
Finger adjustment length 手指調節長度	12.5cm
Adjustable wrist length 手腕調節長度	31cm
Packing size 包裝尺寸	23.5*10*7cm
Package weight 包裝重量	84 g

Instructions

Take out the product and adjust the size of the wrist and finger according to your needs. It can be adjusted appropriately. Put your finger in and use the Velcro to stick the finger part, and then stick the wrist according to the appropriate position, use the reinforcement tape to wrap around the base of the palm and stick the other reinforcement tape around the root of the finger to stick the Velcro to the protector Finished on top

取出產品,根據自己的需求調節腕部大小和手指大小的是適度來調節即可

將手指放進去用魔術貼粘住手指部位,再將腕部根據合適位置來粘緊, 用加固帶在繞手掌根部纏繞粘緊,另一根加固帶繞手指根部纏繞將魔 術貼粘在護具上即完成

www.ziglite.com

Room B3, 18/F Bonsun Industrial Building, 364-366 Sha Tsui Road, Tsuen Wan,HongKong Phone: 852-35863418 Fax: 852-35440462 info@ziglite.com